



Manufactured by
Sunset Healthcare Solutions
180 N Michigan Ave Ste 2000
Chicago, IL 60601



ZCD100KIT-IFU V 1.01
2020-04-20

Instruction Manual

ZOEY

Filter and check valve replacement kit

ZCD100KIT

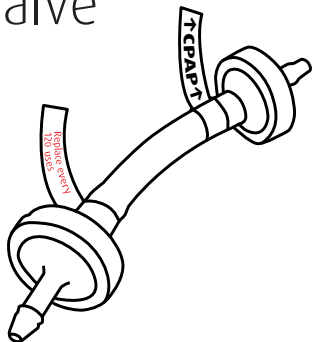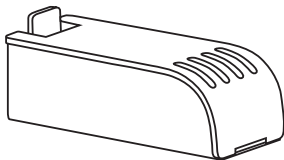

Replace filter



You will need to replace the filter when Zoey displays three solid yellow lights. This will appear at the conclusion of 120 complete cycles.




You **MUST** use the filter to prevent harmful unconverted ozone from entering air.

1

Remove old filter from Zoey's chamber by pushing the filter's release tab and taking filter out of holster.


Dispose of old filter along with household trash.

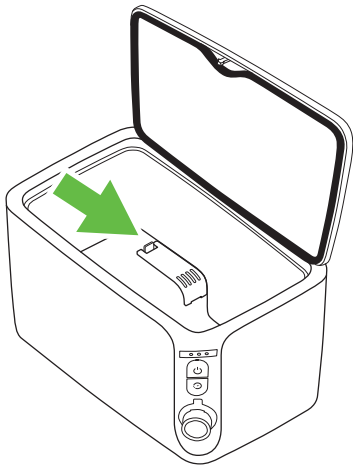

2

Remove replacement filter from plastic wrap. Place into filter holster in Zoey's chamber, vented end first, then click in place.

NOTE: It is normal to find loose charcoal debris in filter plastic wrap.

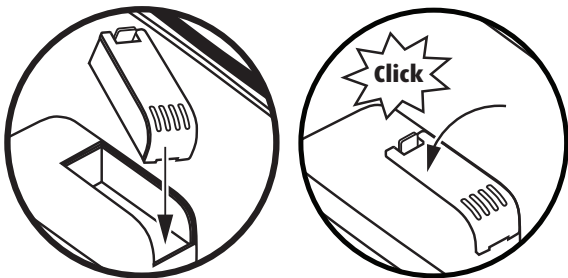

3




After replacing filter, reset Zoey by holding both POWER and DELAY buttons together until green lights are displayed (3 seconds).




This will restart the timer for 120 cycles.
Dispose of old filter along with household trash.

Replace check valve

- 1 Pull thin black injection tubes off of old check valve.

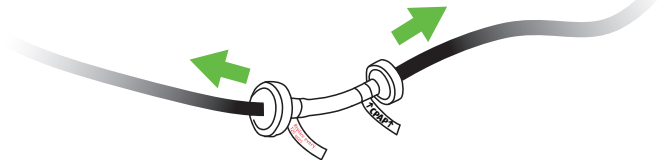

- 2 Put thin black injection tubes onto replacement check valve.

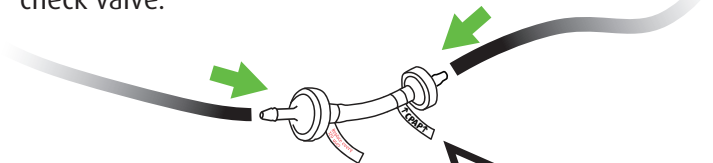

! It is important to ensure the **↑CPAP↑** arrows on the check valve are pointed toward the CPAP tube.

